


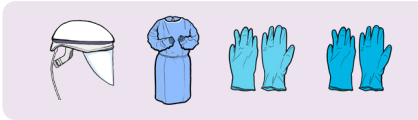

CAPR, launderable gown, double gloves

Checklist

- ① **Batch** work and plan ahead, having items available outside each patient room.
- ② If you already have a mask on: **Sanitize** hands. **Remove** mask. **Store** mask. **Sanitize** hands.
- ③ **Put on** first pair of gloves.
- ④ **Remove** the lens protector liner from the front side of the lens cuff.
- ⑤ **Secure** the lens cuff by **aligning** the center hole over the center attachment post on the helmet.
- ⑥ **Attach** the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet.
- ⑦ **Lift** the cuff flappers to rest against the foam pads at the ends of the comfort strip.
- ⑧ **Insert** the power cord connector into the battery.
- ⑨ **Check** the LED lights to ensure the battery is fully charged.
- ⑩ **Adjust** fan speed to comfort.
- ⑪ **Secure** the battery to the belt using the battery clip.
- ⑫ **Secure** the belt around the waist.
- ⑬ **Turn** ratchet knob counterclockwise to loosen the headband.
- ⑭ **Hold** the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.
- ⑮ **Place** the chin into the lens cuff while pulling the helmet over and down on the head.
- ⑯ **Turn** the ratchet knob counter clockwise to tighten the headband.
- ⑰ **Slide** the fingers around the cuff under the chin to **ensure** slight tension.
- ⑱ **Check** for proper fit, ensure:
 - The cuff flappers are perpendicular to the head.
 - The bottom of the headband rests 1/2 inch from eyebrows.
 - LED indicators are in your peripheral vision.
 - The cuff flappers are 1/4 inch from the temples.
- ⑲ If the headband is not in an appropriate height, doff, **adjust** the headband height position, and redo.
- ⑳ **Put on** gown outside room. Open-end **faces** your back. **Tie** the back of the gown.
- ㉑ **Pull** cuffs half way over palm of the inner glove.
- ㉒ **Put on** second pair of gloves.
- ㉓ **Sanitize** second pair of gloves.
- ㉔ **ENTER room. DO NOT enter** the room if you do not achieve a proper fit of the CAPR system.
- ㉕ **Do not** touch or re-adjust CAPR system inside the room.




CAPR, launderable gown, double gloves

Next patient: **COVID+**


Checklist

Inside room

- ① **Sanitize** outer gloves.
- ② **Remove** outer gloves.
- ③ **Exit** patient room. 

Outside room

- ① **Pull** cuffs half way over palm of the inner glove.
- ② **Put on** fresh outer gloves.
- ③ **Walk** to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.
- ④ **Do not touch** your face, the walls, rails, or anybody else until reaching the next room.
- ⑤ **Sanitize** outer gloves.
- ⑥ **Enter** NEXT COVID-19 patient room or area.
- ⑦ **Do not** touch face or re-adjust CAPR system inside room.

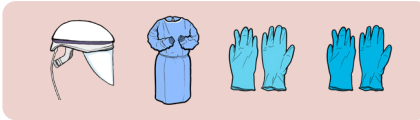


CAPR, launderable gown, double gloves

Next patient: NOT COVID+

Checklist

Inside room

- 1 **Sanitize** outer gloves.
- 2 **Remove** outer gloves.
- 3  **Sanitize** inner gloves.
- 4 **Reach back** and **release** the neck portion.
- 5 **Untie** the waist.
- 6 **Remove** the gown with the outer gloves into the designated bin.
- 7 **Sanitize** inner gloves.
- 8 **Exit** the patient room.

Outside room

- 1 **Sanitize** gloves.
- 2 **Place** first wipe on table.
- 3 **Grab** CAPR lens cuff at the left and right side attachments. **Remove** by pulling outward and forward.
- 4 **Place** CAPR lens cuff on wipe.
- 5 **Wipe** front and back of lens cuff. **Allow** to dry.
- 6 **Store** lens cuff in a gallon sized bag labeled with your name.
- 7 **Sanitize** gloves.
- 8 **Place** second wipe in disinfecting area.
- 9 **Grasp** outer-shell of helmet and remove, **ensure** that the helmet cord doesn't touch your body.
- 10 **Place** helmet down on first wipe.
- 11 **Sanitize** gloves.

- 12 **Put on** mask.
- 13 **Wipe** inside of the helmet.
- 14 **Sanitize** gloves.
- 15 Holding only the inside of the helmet, **flip and transfer** helmet to second wipe.
- 16 **Wipe** outer helmet shell.
- 17 **Sanitize** gloves.
- 18 **Wipe** helmet cord.
- 19 **Sanitize** gloves.
- 20 **Remove** and wipe battery.
- 21 **Sanitize** gloves.
- 22 **Remove** belt and wipe.
- 23 **Sanitize** gloves.
- 24 **Remove** gloves.
- 25 **Wash** hands with soap and water.