


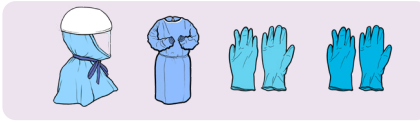

PAPR, launderable gown, double gloves

Checklist

- ① **Batch** work and plan ahead, having items available outside each patient room.
- ② If you already have a mask on: **Sanitize** hands. **Remove** mask. **Store** mask. **Sanitize** hands.
- ③ **Put on** first pair of gloves.
- ④ **Secure** the belt around the waist.
- ⑤ **Secure** the battery on the left side of the belt.
- ⑥ **Connect** the helmet to the battery.
- ⑦ **Check** the LED lights to ensure the battery is fully charged.
- ⑧ **Put** helmet on head and **size** to fit by turning ratchet knob to adjust the headband.
- ⑨ **Adjust** fan speed to comfort.
- ⑩ **Disconnect** the battery.
- ⑪ **Take off** the helmet and **place** it inside the PAPR hood. **Ensure** the white HEPA filter is on top, the helmet is inside filter pouch, and the adjustable knob is outside of filter pouch.
- ⑫ **Slide** the helmet into the hood until it snaps in the center hole. **Snap** helmet into the hood on the left and right side snaps of the face shield.
- ⑬ **Set** the assembled PAPR hood **aside**.
- ⑭ **Sanitize** first pair of gloves.
- ⑮ **Put on** gown outside room. Open-end faces your back. **Tie** gown at the neck.
- ⑯ **Insert** PAPR power cord connector into the battery.
- ⑰ **Tie** gown at waist.
- ⑱ **Put** the helmet on, letting the hood drape around your shoulders over the gown.
- ⑲ **Straighten** the hood draping and **tie** it down.
- ⑳ **Pull** cuffs half way over palm of the inner glove.
- ㉑ **Put on** second pair of gloves.
- ㉒ **Sanitize** second pair of gloves.
- ㉓ **ENTER room. DO NOT enter** the room if you do not achieve a proper fit of the PAPR system.
- ㉔ **Do not** touch or re-adjust PAPR system inside the room.



PAPR, launderable gown, double gloves

Next patient: **COVID+**


Checklist

Inside room

- ① **Sanitize** outer gloves.
- ② **Remove** outer gloves.
- ③ **Exit** the patient room.

Outside room

- ① **Pull** cuffs half way over palm of the inner glove.
- ② **Put on** fresh outer gloves.
- ③ **Walk** to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.
- ④ **Do not touch** your face, the walls, rails, or anybody else until reaching the next room.
- ⑤ **Sanitize** outer gloves.
- ⑥ **Enter** NEXT COVID-19 patient room or area.
- ⑦ **Do not** touch face or re-adjust PAPR system inside room.

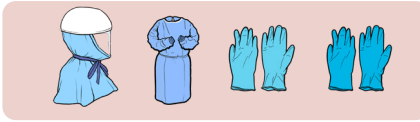

PAPR, launderable gown, double gloves

Next patient: **NOT COVID+**

Checklist

Inside room

- 1 **Sanitize** outer gloves.
- 2 **Remove** outer gloves.
- 3 **Sanitize** inner gloves.
- 4 With your back to the exit, **have buddy stand outside the room and do the next two steps** while wearing clean gloves.
- 5 **Reach back** and **release** the neck portion.
- 6 **Untie** the waist.
- 7 **Remove** the gown with the outer gloves into the designated bin.
- 8 **Sanitize** inner gloves.
- 9 **Exit** the patient room.

Outside room

- 1 **Sanitize** inner gloves.
- 2 **Place** wipe on table.
- 3 **Untie** the PAPR hood.
- 4 **Grasp** face shield at the sides and **pull out** snaps.
- 5 **Pull** face shield forward and **release** helmet pin.
- 6 **Grab** hood top, **pull back**, then **pull forward** to **remove** hood, avoiding your face.
- 7 If reusing, **place** hood in designated bin.
- 8 **Sanitize** gloves.
- 9 **Remove** helmet motor. **DO NOT** touch your face.
- 10 **Put on** mask.
- 11 **Wipe** helmet.
- 12 **Remove** and **wipe** battery.
- 13 **Remove** and **wipe** belt.
- 14 **Sanitize** gloves.
- 15 **Remove** gloves.
- 16 **Wash** hands.