










Inner gloves Outer gloves

Reusable Gown

Gloves

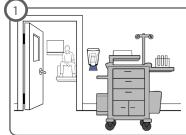

Batch work and **plan ahead**, having items available outside each patient room.






Lens Cuff

Remove the lens protector liner from the front side of the lens cuff.



Secure the lens cuff by aligning the center hole over the center attachment post on the helmet.

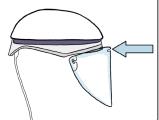


Attach the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet.

Tip: The lens cuff will click when it is properly secured.




Lift the cuff flappers to rest against the foam pads at the ends of the comfort strip.

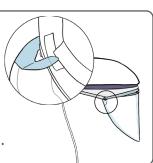

Insert
the power
cord
connecter
into the
battery.

Check the LED lights to ensure the battery is fully charged.

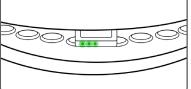


Adjust fan speed to comfort.

Battery and Belt

Secure the battery to the belt using the battery clip.



Secure the belt around the waist.

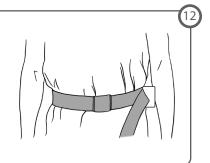










CAPR, gown, double gloves

Helmet

Turn ratchet knob counterclockwise to loosen the headband.



Hold the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.

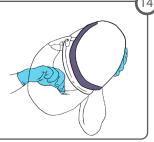

Place the chin into the lens cuff while pulling the helmet over and down on the head.



Turn the ratchet knob counterclockwise to tighten the headband.



Slide the fingers around the cuff under the chin to ensure slight tension.




18 **Check** for proper fit, ensure:

> The cuff flappers are perpendicular to the head

The bottom of the headband rests 1/2 inch from the eyebrows



LED indicators are in your

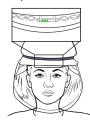

The cuff flappers are 1/4 inch peripheral vision. from the temples



Tip: If the headband is not in an appropriate height, doff, adjust the headband height position, and redo



Entry



Put on gown outside room.

Open-end **faces** your back.

Tie the back of the gown.



Put on second pair of gloves.



Sanitize second pair of gloves.



ENTER room

DO NOT enter the room if you do not achieve a proper fit of the CAPR system. _



Do not touch or re-adjust CAPR system inside the room.













Next patient: COVID+

Patient Room







Exit











OUTSIDE

Next patient: COVID+


Gloves

After outer glove removal in the patient room, exit and...





Walking

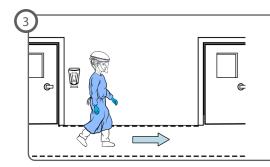

Walk to next patient with confirmed COVID-19 (marked) while remaining in marked zone.



Do not touch your face, the walls, rails, or anybody else until reaching the next room.

Entry

















Next patient: NOT COVID+

Coveralls












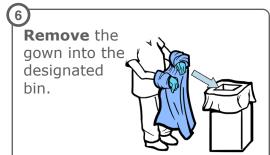



Exit











OUTSIDE room

CAPR, gown, double gloves

Next patient: **NOT** COVID+

Lens Cuff

After coveralls and outer glove removal...





Grab lens cuff at the left and right side attachments. **Remove** and by pulling outward and forward.







Store lens cuff in a gallon sized bag labeled with vour name.



Sanitize gloves.




Place second wipe on table.



Helmet

Grasp outer-shell of helmet and take off.

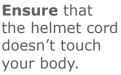




Place helmet on first wipe.








Hold inside of the helmet and flip onto second wipe.

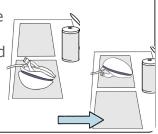

Wipe outside of the helmet.

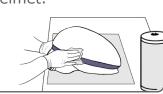









OUTSIDE room

Next patient: **NOT** COVID+

Battery













Hands









