

putting PPE **ON**



Coveralls CAPR system Inner gloves Outer gloves

What You Need

1 of 2

Gloves

1 **Batch work and plan ahead**, having items available outside each patient room.

2 **Sanitize hands.**

3 **Put on first pair of gloves.**

Lens Cuff

4 **Remove** the lens protector liner from the front side of the lens cuff.

5 **Secure** the lens cuff by **aligning** the center hole over the center attachment post on the helmet.

6 **Attach** the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet.

Tip: The lens cuff will click when it is properly secured.

7 **Lift** the cuff flappers to rest against the foam pads at the ends of the comfort strip.

8 **Insert** the power cord connector into the battery.

9 **Check** the LED lights to ensure the battery is fully charged.

10 **Adjust** fan speed to comfort.

Battery and Belt

11 **Secure** the battery to the belt using the battery clip.

12 **Secure** the belt around the waist.

putting PPE **ON**



Helmet


13

Turn ratchet knob counterclockwise to loosen the headband.



14

Hold the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.

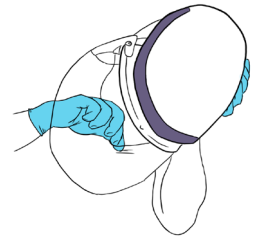

15

Place the chin into the lens cuff while pulling the helmet over and down on the head.



16

Turn the ratchet knob counter clockwise to tighten the headband.



17

Slide the fingers around the cuff under the chin to **ensure** slight tension.




18


Check for proper fit, ensure:

The cuff flappers are perpendicular to the head

The bottom of the headband rests 1/2 inch from eyebrows

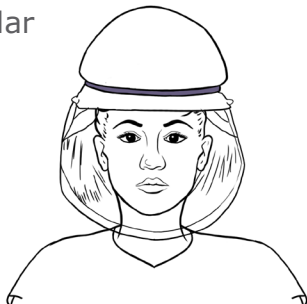

LED indicators are in your peripheral vision.

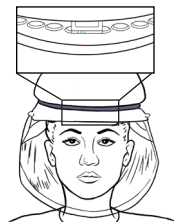


The cuff flappers are 1/4 inch from the temples



19

Tip: If the headband is not in an appropriate height, doff, adjust the headband height position, and redo

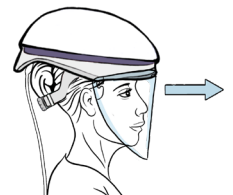

Entry

20



Put on coveralls outside room.
Open-end **faces** your front.
Zip up the front of the coveralls.

21

Sanitize first pair of gloves.



22

Put on second pair of gloves.



23

Sanitize second pair of gloves.



24

ENTER room

DO NOT enter the room if you do not achieve a proper fit of the CAPR system.



25

Do not touch or re-adjust CAPR system inside the room.

