

taking PPE

OFF



(PAPR)

OUTSIDE ROOM

# TERMINAL DOFFING

Next patient: **NOT COVID+**

## Hood

After coveralls and outer glove removal...

- 1 **Sanitize** gloves
- 2 **Place** first wipe on table.
- 3 **Untie** the PAPR hood.
- 4 **Grasp** face shield at the sides and **pull out** snaps.
- 5 **Pull** face shield forward and **release** helmet pin.
- 6 **Grab** top of hood and **pull back**, then **pull forward** to **remove** the hood.
- 7 If reusing, **place** hood in designated bin.

## Helmet, Belt, Battery

- 8 **Sanitize** gloves.
- 9 **Remove** helmet motor, **DO NOT** touch your face.
- 10 **Wipe** helmet.
- 11 **Remove** and **wipe** battery.
- 12 **Remove** and **wipe** belt.
- 13 **Sanitize** gloves.
- 14 **Remove** gloves.
- 15 **Wash** hands.