

putting PPE ON ^{1/2}



Reusable Gown



CAPR system



Inner gloves



Outer gloves

Gloves

1 **Batch work and plan ahead**, having items available outside each patient room.

2 **Sanitize hands.**

3 **Put on first pair of gloves.**

Lens Cuff

4 **Remove** the lens protector liner from the front side of the lens cuff.

5 **Secure** the lens cuff by **aligning** the center hole over the center attachment post on the helmet.

6 **Attach** the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet.

Tip: The lens cuff will click when it is properly secured.

7 **Lift** the cuff flappers to rest against the foam pads at the ends of the comfort strip.

8 **Insert** the power cord connecter into the battery.


9 **Check** the LED lights to ensure the battery is fully charged.

10 **Adjust** fan speed to comfort.

Battery and Belt

11 **Secure** the battery to the belt using the battery clip.

12 **Secure** the belt around the waist.

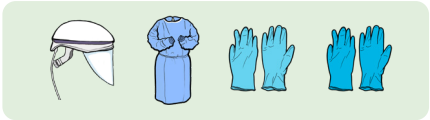

CAPR, gown, double gloves

Helmet

13 **Turn** ratchet knob counterclockwise to loosen the headband.

14 **Hold** the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.

15 **Place** the chin into the lens cuff while pulling the helmet over and down on the head.

16 **Turn** the ratchet knob counter clockwise to tighten the headband.

17 **Slide** the fingers around the cuff under the chin to **ensure** slight tension.

18 **Check** for proper fit, ensure:

- The cuff flappers are perpendicular to the head
- The bottom of the headband rests 1/2 inch from eyebrows
- LED indicators are in your peripheral vision.
- The cuff flappers are 1/4 inch from the temples

19 **Tip:** If the headband is not in an appropriate height, doff, adjust the headband height position, and redo

Entry

20 **Put on** gown outside room. Open-end **faces** your back. **Tie** the back of the gown.


21 **Sanitize** first pair of gloves.

22 **Put on** second pair of gloves.

23 **Sanitize** second pair of gloves.

24 **ENTER room**
DO NOT enter the room if you do not achieve a proper fit of the CAPR system.

25 **Do not touch or re-adjust CAPR system** inside the room.

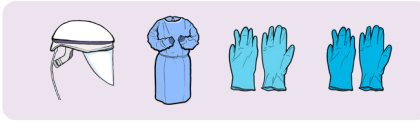

Next patient: COVID+

Patient Room

1 **Sanitize** outer gloves.

A hand sanitizer dispenser with a blue liquid inside. A pair of hands wearing blue gloves is being held under the dispenser's nozzle.

2 **Remove** outer gloves.

A pair of hands wearing blue gloves. One hand is being pulled away from the other, showing the wrist and cuff area.


3 **Sanitize** inner gloves.

A hand sanitizer dispenser with a blue liquid inside. A pair of hands wearing blue gloves is being held under the dispenser's nozzle.

Exit

4 **Exit** patient room.

A simple line drawing of an exit door. Above the door is a sign that says "EXIT". To the left of the door is a hand sanitizer dispenser.

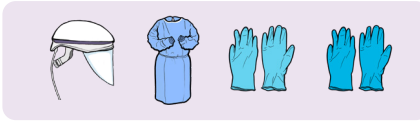

CAPR, gown, double gloves

Next patient: COVID+

Gloves

After outer glove removal in the patient room, exit and...

1 **Sanitize** inner gloves.

A pair of blue-gloved hands is being held under a white hand sanitizer dispenser.

2 **Put on** fresh outer gloves.

A hand is shown pulling a new blue glove onto the wrist.

Walking

3 **Walk** to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.

A person in full PPE is walking in a hallway between two doors. A dashed line and an arrow indicate a marked path.

4 **Do not touch** your face, the walls, rails, or anybody else until reaching the next room.

A person in full PPE is walking in a hallway. Red X's are placed over their face, hands touching the wall, and hands touching another person.

Entry

5 **Sanitize** outer gloves.


A pair of blue-gloved hands is being held under a white hand sanitizer dispenser.

6 **Enter** NEXT COVID-19 patient room or area.

A person in full PPE is walking through a doorway into a room. A sign above the doorway says "ENTER".

7 **Do not touch** face or re-adjust CAPR system inside room.

A person's face is shown with red X's over their eyes, nose, and the CAPR system.

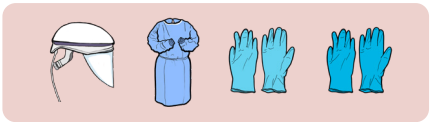

CAPR, gown, double gloves

Next patient: NOT COVID+

Coveralls

1 **Sanitize** outer gloves.

A hand sanitizer dispenser with a hand being inserted into the nozzle.

2 **Remove** outer gloves.

A pair of hands pulling off a pair of blue gloves.

3 **Sanitize** inner gloves.

A hand sanitizer dispenser with a hand being inserted into the nozzle.

4 **Reach back** and **release** the neck portion.

A person in a blue gown reaching back to untie the neck portion.

5 **Untie** the waist.

A person in a blue gown untieing the waist portion.

6 **Remove** into bin for reuse without touching outside of gown.

A person in a blue gown placing the gown into a bin.

7 **Sanitize** inner gloves.


A hand sanitizer dispenser with a hand being inserted into the nozzle.

Exit

8 **Exit** patient room.

A person in a blue gown exiting a patient room through a door labeled 'EXIT'.

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.

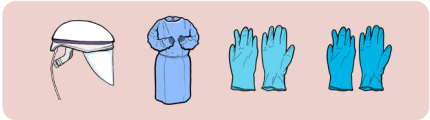

CAPR, gown, double gloves

Next patient: NOT COVID+

Lens Cuff

After coveralls and outer glove removal...

1 **Sanitize** gloves

2 **Place** first wipe on table.

3 **Grab** lens cuff at the left and right side attachments. **Remove** and by pulling outward and forward.

4 **Place** lens cuff on wipe.

5 **Wipe** front and back of lens cuff. **Allow** to dry.

6 **Store** lens cuff in a gallon sized bag labeled with your name.

Helmet

7 **Sanitize** gloves.

8 **Place** second wipe on table.

9 Grasp outer-shell of helmet and **take off**. **Ensure** that the helmet cord doesn't touch your body.

10 **Place** helmet on first wipe.

11 **Sanitize** gloves.


12 **Wipe** inside of the helmet.

13 **Sanitize** gloves.

14 Hold inside of the helmet and **flip** onto second wipe.

15 **Wipe** outside of the helmet.

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.

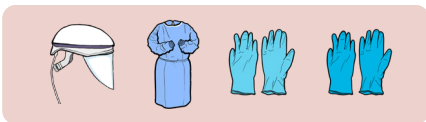

CAPR, gown, double gloves

Next patient: NOT COVID+

Battery

16



Sanitize gloves.

17

Wipe helmet cord.



18



Sanitize gloves.

19

Remove and wipe battery.



20



Sanitize gloves.

21

Remove belt and wipe.



Hands

22



Sanitize gloves.

23

Remove gloves.



24

Wash hands.

